

# QuikClot<sup>®</sup> COMBAT GAUZE<sup>®</sup> XL



**Vacuum Packed and Z-Folded**

**For Temporary External Use To Control Traumatic Bleeding**

**ONE (1) 2-PLY 4 IN X 4 YDS (10.2 CM X 3.7 M) HEMOSTATIC DRESSING**

**NSN 6510-01-600-4705**



Made in the USA

## DIRECTIONS FOR USE



1. Open package and remove **Combat Gauze XL**. Keep the empty package.



2. Pack **Combat Gauze XL** into wound and use it to apply pressure directly over bleeding source. More than one **Combat Gauze XL** may be required.



3. Continue to apply pressure for 3 minutes or until bleeding stops.



4. Wrap and tie bandage to maintain pressure. Seek medical care immediately. Show **PRODUCT REMOVAL** directions on package to medical personnel.

### PRODUCT REMOVAL:

1. Gently remove gauze from wound.
2. Thoroughly irrigate wound.



Do Not Reuse



STERILE

SBS

Sterile Barrier System

STERILE R

MD Medical Device

**WARNINGS:** For external use only. Sterility not guaranteed if package is damaged or opened. Discard package if damaged. Avoid contact with eyes. Reuse will cause risk of infection and loss of efficacy.

## FOR USE BY TRAINED EMERGENCY RESPONDERS

Manufactured by:  
Z-Medica<sup>®</sup>, LLC  
4 Fairfield Boulevard  
Wallingford, CT 06492 USA  
Tel: 1.877.750.0504, +1.203.294.0000  
Fax: 1.800.343.8656, +1.203.303.7216  
QuikClot.com  
Patents: Z-Medica.com/patents.aspx  
© 2020 Z-Medica, LLC  
All rights reserved.  
Contains kaolin.  
100987-01-WEB Rev I 10/9/2020

CE

2797

EC REP

Emergo Europe  
Prinsessegracht 20  
2514 AP  
The Hague  
The Netherlands

REF 345