



CONTROL+[®]

REF 4030

HEMOSTATIC DRESSING, 12 in x 12 in 3-Ply Pad Dressing

Description

QuikClot Control+[®] consists of a white to off-white to yellow, sterile, 3-ply, 12 in x 12 in, X-Ray detectable hemostatic dressing and is packaged for aseptic removal.

Indications

QuikClot Control+ is indicated for temporary control of internal organ space bleeding for patients displaying class III or class IV bleeding. It may also be used for control of severely bleeding wounds such as surgical wounds and traumatic injuries.

Contraindications

- Do not leave QuikClot Control+ in place for more than 48 hours.
- QuikClot Control+ is not indicated for intraluminal vascular use.

Warnings

- Adhesion formation associated with QuikClot Control+ dressing use was noted in preclinical studies; adhesions were also observed with control materials. It is not known whether adhesions elicited by QuikClot Control+ dressing are equivalent to those caused by control materials.
- QuikClot Control+ dressing is not absorbable and must be removed from the wound prior to wound closure.

Precautions

- If bleeding persists, additional product may be applied to the wound.

Directions for Use

1. Verify the expiration date on the package labels prior to using the product. Open package and remove QuikClot Control+ dressing.
2. Apply QuikClot Control+ dressing to the wound and apply pressure until bleeding is controlled. More than one dressing may be required.
NOTE: If needed, additional gauze or a pressure dressing may be applied to maintain pressure. The time for formation of a stable clot may vary depending on several patient factors.
3. Remove the dressing and repair the wound, if necessary. If the dressing is adhered to the wound, hydrate with sterile saline to aid in removal.
4. Copiously irrigate the QuikClot Control+ dressing application sites and completely suction irrigation fluid prior to wound closure.

Storage Conditions

Keep dry. Keep away from heat, including storage in direct sunlight or in direct contact with heat sources.

Sterility and Expiration

Product is sterilized by exposure to gamma radiation and is intended for single use only. Do not attempt to resterilize the device by any means. Do not use if the sterile pouch is damaged or opened. Do not use the device after the expiration date listed on the package.

Rx ONLY

Manufactured by:

Z-Medica[®], LLC

4 Fairfield Boulevard

Wallingford, CT 06492 USA

Tel: 1-877-750-0504, +1-203-294-0000

Fax: 1-800-343-8656, +1-203-303-7216

QuikClotControlPlus.com

Patents: Z-Medica.com/patents.aspx

© 2020 Z-Medica, LLC

All rights reserved.

Made in the USA

101620-01-WEB Rev C 10/13/2020

Please contact Z-Medica Customer Service to determine if this product is available for sale in your market / country.